

# Stay home and sigvaris

## Yes, You Can!

We understand that taking care of your graduated compression products may seem intimidating, but it doesn't have to be! We decided to answer some of your frequently asked questions so taking care of your compression products is easier than ever:



**Yes, you can** wash your garments like you would any other blouse or button down

**Yes, you can** use your favorite, everyday detergent, just make sure it is free of bleach and fabric softener



**Yes, you can** dry your socks and hosiery in the dryer – dry them on a low heat setting like any other delicate garment



If the grip-top on your product loses its hold, simply wash the adhesive band with warm water and gentle soap or wipe it with rubbing alcohol.

Now, all you'll have to worry about is which pair of your Sigvaris socks and hosiery you're going to wear today!

If you have any questions, please reach out to Customer Care:

**1-800-322-7744, ext. 1.**

Live your life.  
Be yourself.

SIGVARIS  
GROUP